

Read the whole package insert carefully: it contains important information for the consumer. This is a medical device which is available without medical prescription. However, in order to get a better efficacy, it should be used according to the described indications.

- Keep this package insert, you may need to read it again.
- For further information or advice, ask your doctor and/or pharmacist.
- If the symptomatology persists after a certain time, consult your doctor.

Cicatridina®

HYALURONIC ACID SODIUM SALT 5 mg

vaginal ovules

COMPOSITION

Hyaluronic acid sodium salt 5 mg, *Centella asiatica* oil extract, Calendula oil extract, Aloe vera oil extract, Tea tree essential oil, semi-synthetic glycerides, BHT.

HOW IT IS PRESENTED

CICATRIDINA® is presented in the form of ovules for vaginal use. The pack contains 10 vaginal ovules of 2 g e.

WHAT IT IS AND WHY IT IS USED

The activity of CICATRIDINA® vaginal ovules is due to the presence of hyaluronic acid, a mucopolysaccharide which is abundantly distributed in many body tissues. Hyaluronic acid functions as structural support giving tonicity, trophism and elasticity to tissue. Numerous clinical studies have shown the capacity of hyaluronic acid to accelerate and improve the process of re-epithelialisation and tissue repair.

INDICATIONS

Adjuvant treatment of reparative processes in atrophic and dystrophic conditions of the vaginal mucosa. It promotes healing after childbirth, in gynaecological surgery, in cases of dystrophy resulting from chemotherapy, ionising radiation, vaginal dryness also due to oestrogen deficiency.

WHEN IT SHOULD NOT BE USED

In case of individual hypersensitivity to any of the components of the product.

Pregnancy and breast feeding

The safety of use of CICATRIDINA® vaginal ovules during pregnancy and breast feeding has not been demonstrated. In such circumstances, it is advisable to consult your doctor.

Effects on ability to drive and use equipments or machines

No special precautions are necessary.

INTERACTIONS WITH OTHER PRODUCTS

Interactions with other products are not known; therefore, it is advisable to consult the doctor before using CICATRIDINA® vaginal ovules in association with other medical devices or drugs for vaginal use.

HOW TO USE IT

Remove the ovule by pulling on the two pre-cut flyers of the blister and insert it deep into the vagina.

Wash hands before and after the use of the medical device.



WHEN AND FOR HOW LONG

Use one ovule in the evening or according to medical prescription. The duration of use depends on the evolution of symptoms; if necessary, CICATRIDINA® vaginal ovules may be used for long periods.

UNDESIRABLE EFFECTS

The use of all topical products, especially if prolonged, may cause sensitization. If this occurs, stop treatment and consult a doctor to start suitable therapy. If undesired effects not described in this package insert occur, inform your doctor or pharmacist.

WARNINGS

Keep the product out of the reach of children. Avoid contact with eyes. Do not swallow.

EXPIRY AND STORAGE

The expiry date is stated on the pack (EXP) and it refers to the product in an unopened and correctly stored pack. Do not use the product after the date stated on the pack, or if the pack is open or damaged. Store the medical device tightly closed, away from light and heat, preferably at a temperature between 4 °C and 25 °C.

MEDICAL DEVICE

CE 0373



Manufacturer:
FARMA-DERMA s.r.l.

Via dei Bersaglieri, 10 - 40010 Sala Bolognese (BO) ITALY