


INSTRUCTIONS FOR USING THE VICTOZA® PEN


Please read these instructions carefully before using your Victoza® pen.

Your Victoza® pen comes with 18 mg of liraglutide. You can select doses of 0.6 mg, 1.2 mg and 1.8 mg. Victoza® pen is designed to be used with NovoFine® or NovoTwist® disposable injection needles up to a length of 8 mm and as thin as 32G.

Victoza® pen

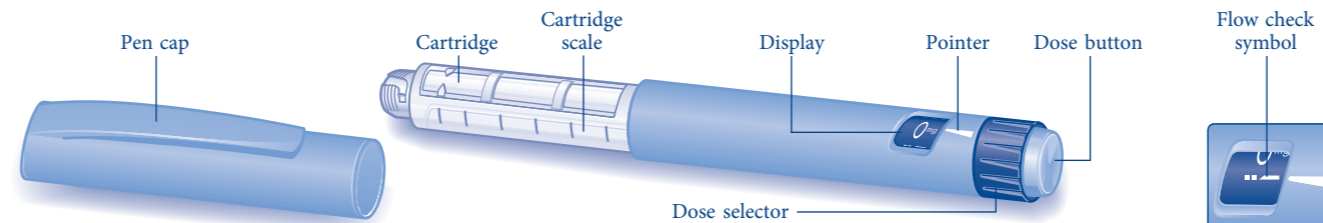


Needle (example)

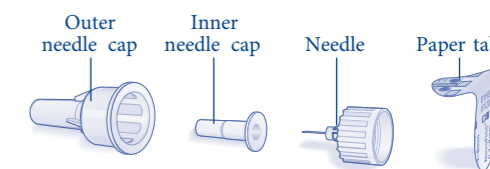


Preparing your Victoza® pen


A Pull off the pen cap.

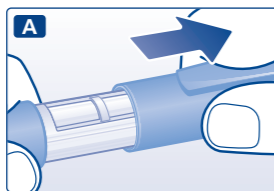


B Pull off the paper tab from a new disposable needle. Screw the needle straight and tightly onto your pen.

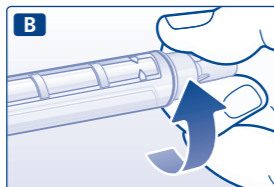

C Pull off the outer needle cap and keep it for later.

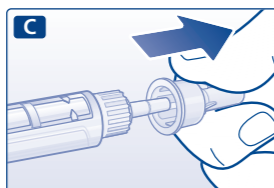

D Pull off the inner needle cap and throw it away.

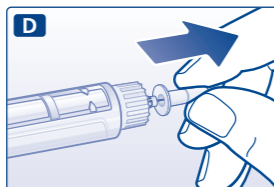


- △ Always use a new needle for each injection to prevent contamination.
- △ Be careful not to bend or damage the needle.
- △ Never put the inner needle cap back on when you have removed it from the needle. This reduces the risk of hurting yourself with the needle.

Checking the flow

Always check the flow as follows before you inject with a new pen.

E Turn the dose selector until the flow check symbol lines up with the pointer.

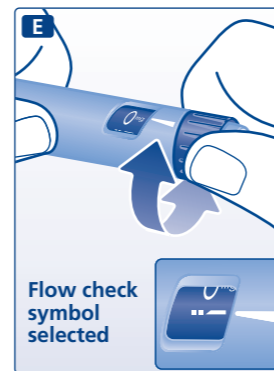

F Hold the pen with the needle pointing up. Tap the cartridge gently with your finger a few times. This will make any air bubbles collect at the top of the cartridge.



G Keep the needle pointing up and press the dose button until 0 mg lines up with the pointer. Repeat steps **E** to **G** until a drop of liraglutide appears at the needle tip. If no drop appears after six times, change the needle and repeat steps **E** to **G** up to six more times. If you still see no drop of liraglutide, the pen is broken and you must use a new one.



- △ If you have dropped your pen against a hard surface or suspect that something is wrong with it, always put on a new disposable needle and check the flow before you inject.

Selecting your dose

Always check that the pointer lines up with 0 mg.

H Turn the dose selector until your needed dose lines up with the pointer (0.6 mg, 1.2 mg or 1.8 mg). If you selected a wrong dose by mistake, simply change it by turning the dose selector backwards or forwards until the right dose lines up with the pointer. Be careful not to press the dose button when turning the dose selector backwards, as liraglutide may come out. If the dose selector stops before your needed dose lines up with the pointer, there is not enough liraglutide left for a full dose. Then you can either:



Divide your dose into two injections:

Turn the dose selector in either direction until 0.6 mg or 1.2 mg lines up with the pointer. Inject the dose. Prepare a new pen for injection and inject the remaining number of mg to complete your dose.

Inject the full dose with a new pen:


If the dose selector stops before 0.6 mg lines up with the pointer, prepare a new pen and inject the full dose with the new pen.

- △ The dose selector clicks when you turn it. You must not use these clicks to select the amount of liraglutide to inject.
- △ Do not use the cartridge scale to measure how much liraglutide to inject – it is not accurate enough.
- △ Do not try to select other doses than 0.6 mg, 1.2 mg or 1.8 mg. The numbers in the display must line up precisely with the pointer to ensure that you get a correct dose.


Using your injection

Insert the needle into your skin using the injection technique shown by your doctor or nurse. Then follow the instructions below:


I Press the dose button to inject until 0 mg lines up with the pointer. Be careful not to touch the display with your other fingers or press the dose selector sideways when you inject. This is because it may block the injection. Keep the dose button pressed down and leave the needle under the skin for at least six seconds. This is to make sure that you get your full dose.

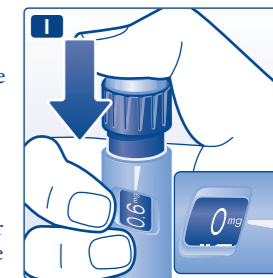


J Pull out the needle. After that, you may see a drop of liraglutide at the needle tip. This is normal and has no effect on the dose you have just had.

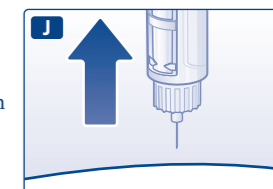

K Guide the needle tip into the outer needle cap without touching the outer needle cap.

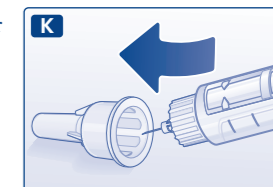

L When the needle is covered, carefully push the outer needle cap completely on. Then unscrew the needle. Carefully throw the needle away and put the pen cap back on. When the pen is empty, carefully throw it away without a needle attached. Please throw the pen and needle away in accordance with local requirements.

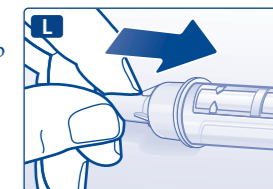

Caring for your Victoza® pen

Your Victoza® pen is accurate and safe to use. But you must take care of it:

- Do not try to repair your pen or pull it apart.
- Keep your pen away from dust, dirt and all kinds of liquids.
- Clean the pen with a cloth moistened with a mild detergent. Do not try to wash it, soak it or lubricate it – this can harm the pen.

△ Important information

- Do not share your Victoza® pen with anyone else.
- Keep your Victoza® pen out of reach of others, especially children.

- △ Always remove the needle after each injection and store your Victoza® pen without a needle attached.
- △ This prevents contamination or infection or leakage of liraglutide. It also ensures that the dosing is accurate.
- △ Caregivers should be very careful when handling used needles to avoid hurting themselves with the needles.