
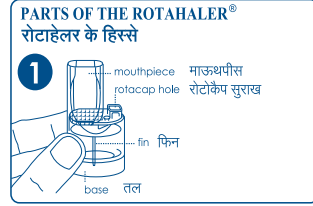
## HOW TO USE ROTACAPS WITH THE ROTAHALER®

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
**1 :** Hold the Rotahaler vertically as shown. Note: It is important to position the two halves of the Rotahaler such that the fin is not directly below the Rotacap hole.

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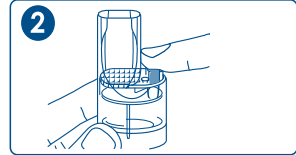
**2 :** Take a Rotacap from its bottle. Insert the Rotacap into the Rotacap hole with the transparent end facing down as shown. Press the Rotacap firmly so that the top end is level with the top of the Rotacap hole.

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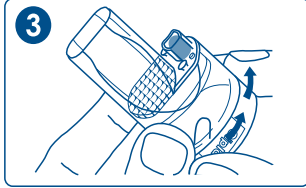**3 :** Holding the mouthpiece firmly with one hand, rotate the base. The fin separates the two halves of the Rotacap as can be seen through the transparent body of the Rotahaler. The Rotahaler is now ready for use.

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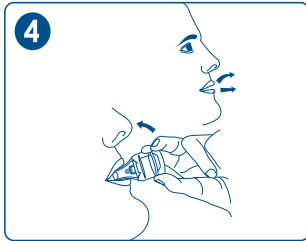**4 :** Breathe out fully through your mouth. Position the mouthpiece between your teeth and close your lips tightly around it.

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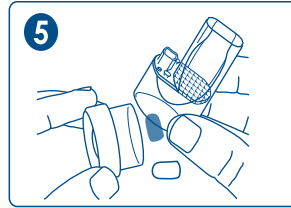
Remove the Rotahaler from your mouth and hold your breath for about 10 seconds or for as long as it is comfortable. It is important to inhale all the powder. You may need to repeat step 4 to ensure this.

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
If you are taking a second rotacap insert a new Rotacap as shown in Step 2. When the new rotacap is inserted the empty rotacap remaining in the Rotacap hole will be pushed out. Proceed with steps 3 & 4.

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**5 :** After use, separate the two halves of the Rotahaler and discard the empty rotacap(s). Rejoin the two halves of the Rotahaler to be ready for the next use.

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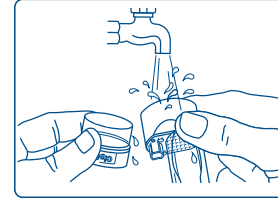### CLEANING THE ROTAHALER®

Wash the Rotahaler at least once a week:

- Separate the two halves of the Rotahaler.
- Remove the empty Rotacap from the Rotacap hole (if any) and discard the empty Rotacaps.
- Rinse the two halves in clean running water.
- Shake well to remove excess water and leave the two halves to air dry. Do not expose the Rotahaler to heat.
- Rejoin the two halves of the Rotahaler to be ready for the next use.

**CAUTION:**  
DO NOT PUSH ANY CLOTH OR INSTRUMENT INTO THE MOUTHPIECE AS THIS MAY DAMAGE THE ROTAHALER

### REMEMBER

- Keep the Rotahaler clean and dry at all times. Never use the Rotahaler when it is wet.
- It is recommended to use a new Rotahaler every 6 months.
- Use only as directed by your doctor. If the recommended dose does not provide relief from symptoms or if symptoms become worse, consult your doctor.
- Keep the Rotacap container tightly closed. Insert the Rotacap into the Rotahaler just

prior to use as capsules exposed to moisture may not break open.

- At times a fine layer of powder may remain in the empty Rotacap after use. This is normal and does not alter the effect of the medicine.
- Store the Rotahaler along with the Rotacap bottle in the convenient carry box provided.

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21060765

**Cipla**

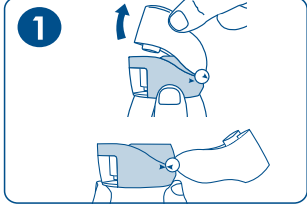
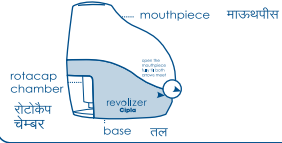
## HOW TO USE ROTACAPS WITH THE REVOLIZER®

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

1 : To open, hold the Revolizer at the base with one hand, and pull back the mouthpiece as shown till both arrows meet.

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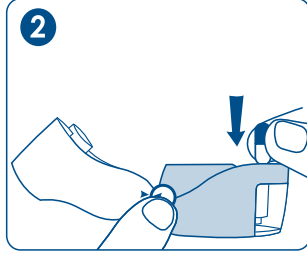
2 : Take a Rotacap from its bottle. Insert the Rotacap into the Rotacap chamber with the transparent end facing down as shown.

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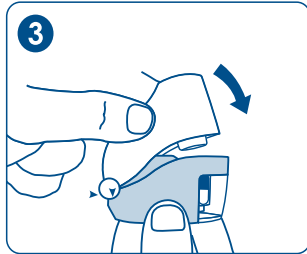3 : Close the mouthpiece firmly. A 'click' sound will indicate proper closing of the Revolizer.

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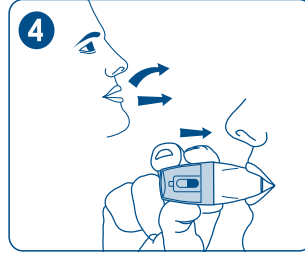4 : Breathe out fully through your mouth. Position the mouthpiece between your teeth and close your lips tightly around it.

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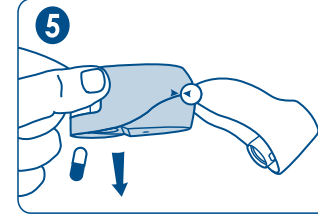
Sit or stand upright, keep your head straight, and breathe in through your mouth as rapidly and deeply as you can. When done correctly, you will hear the Rotacap vibrating inside the Revolizer.

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
Remove the Revolizer from your mouth and hold your breath for about 10 seconds or for as long as it is comfortable. It is important to inhale all the powder which can be seen through the Rotacap chamber. You may need to repeat step 4 to ensure this.

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5 : After every use, open the mouthpiece (till both arrows meet) and discard the empty Rotacap. Close the mouthpiece and store the Revolizer in the convenient carry pouch provided. For your next use, take a new Rotacap and follow steps 1 to 5.

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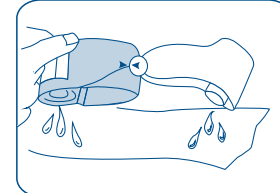### CLEANING THE REVOLIZER® You must remember to clean the Revolizer at least once a week to ensure smooth functioning.

- Open the mouthpiece (till both arrows meet) of the Revolizer.
- Remove the empty Rotacap (if any) from the Rotacap chamber
- Rinse the mouthpiece and Rotacap chamber in clean running water.
- Shake well to remove excess water and leave to air dry. This can take upto 6 hours, so preferably clean the

Revolizer and leave to dry overnight. Do not expose the Revolizer to heat.

**CAUTION:**  
DO NOT PUSH ANY CLOTH OR INSTRUMENT INTO THE MOUTHPIECE OF THE REVOLIZER AS THIS MAY CAUSE DAMAGE

#### REMEMBER:

- Keep the Revolizer clean and dry at all times. Never use the Revolizer when it is wet.
- It is recommended to use a new Revolizer every 6 months.
- Use only as directed by your doctor. If the recommended dose does not provide relief from symptoms or if symptoms become worse, consult your doctor
- Keep the Rotacap bottle tightly closed to avoid exposure to moisture. Open the Rotacap bottle just prior to use.
- At times a fine layer of powder may remain in the empty Rotacap after use. This is normal and does not alter the effect of the medicine.
- Store the Revolizer along with the Rotacap bottle in the convenient carry pouch provided.

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