UNIMER[®]

PEDIATRIC

150 tonic 9%

SEAWATER

FR

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FR

Effective nasal hygiene encourages protection of the nasal mucosa against outside attacks (cold, allergens, dry atmospheres, pollution, etc.) and helps the nose ensure its respiratory function.

Up to 3 years, children do not know how to blow their nose and are reticent towards daily nasal hygiene. LDPSA has developed UNIMER PEDIATRIC ISOTONIC with an entertaining and reassuring design, which helps children learning how to blow their nose in a fun way, without being afraid of the spray.

UNIMER PEDIATRIC ISOTONIC is a purified, microbiologically-controlled isotonic seawater solution.

UNIMER PEDIATRIC ISOTONIC can be used:

- To ensure effective nasal hygiene,
- To encourage prevention of rhino-pharyngeal infections,
- To remove mucosity in obstructed nose during colds,
- To ensure protection of the nasal mucosa against attacks: cold, pollution, dust, pollen, etc.

Composition:

Isotonic seawater solution, 9 g/L in NaCl. Inert propellant gas: nitrogen, without contact with the product.

Preservative-free.

Posology:

UNIMER PEDIATRIC ISOTONIC can be used from birth thanks to its micro-diffusion and safety nozzle, tailored to the anatomy of baby's nose. Administer 1 to 2 brief sprays in each nostril, preferably before meals, 2 to 6 times per day or according to your doctor's advice.

Directions for Use:

Place the nozzle to the bottle. Tilt the head to the side, and insert the nozzle gently into the upper nostril. Press briefly, then repeat on the other side. See images.

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