

irilens®

**For Dry Eyes
For Contact Lens Wearers**

**HYDRATING AND LUBRICATING
DRY EYES**

10 mL Bottle

Irilens eye drops is a sterile ophthalmic solution with hydrating and lubricating properties. It contains high molecular weight sodium hyaluronate 0.2% of biotechnological origin and sodium hydroxymethylglycinate (SHMG) as a preservative. Contains no benzalkonium chloride. Also suitable for contact lens wearers. Sodium hyaluronate is a mucopolysaccharide which restores balance in case of changes in the tear film.

Intended Use: Irilens eye drops is a lubricating and hydrating tear substitute indicated for all cases of discomfort caused by dry eyes. The most common factors leading to dry eyes may include:

- Contact lens wear.
- Environmental factors: sun, wind, cold, smoke filled rooms, dry air from indoor heating and cooling.
- Prolonged computer use.
- Ageing.
- Drug therapy: antidepressants, anxiolytics, hormones, and antihistamines.
- Eye surgery.

Instructions for Use:

- Open the bottle.
- Apply 1-2 drops in each eye while lightly squeezing the bottle.
- Close the container immediately after use: the contents of the bottle can be used for up to 28 days after first opening.
- The bottled product is for single patient use only.
- If necessary, repeat the above steps 2-4 times daily.
- Contact lens wearers can apply 1-2 drops of solution to each lens before inserting contact lenses, in order to keep the eye moist and lubricated.

Composition: Sodium hyaluronate 0.2%, sodium hydroxymethylglycinate, sodium phosphate monobasic monohydrate, sodium phosphate dibasic dodecahydrate, sodium chloride, EDTA, water for injection.

Precautions and Warnings: This product is for external ophthalmic use only.

- Do not use this product if the container appears damaged or is not perfectly closed at the time of first opening.
- Wait at least 15 minutes after use before applying medicinal products for external ophthalmic use.
- Do not touch the eye or other surfaces with the tip of the dispenser during the application.
- Do not use in case of known incompatibility with any component of the preparation.
- Store below 30°C.
- Do not use this product after the expiry date which is stated on the package.
- Dispose of the container properly after use.
- Keep this product out of reach of children.
- In case of discomfort during the use of this product, stop treatment and contact a doctor.
- In some cases, short term blurring of vision has been observed after administration due to the density of the solution, but this resolves very rapidly.

Possible Side Effects: No side effects have been reported to date.

Package Size: Multidose bottle containing 10 mL of sterile solution.

Shelf Life: Unopened package: do not exceed the expiry date which is stated on the carton and bottle.

Date of Last Revision: February 2013



COC Farmaceutici s.r.l.

Via Modena, 15 - 40019 Sant'Agata Bolognese (BO)

Exclusive Distributor: Biofinpharma Middle East
26th Street - Bldg 10 - North Metn - Rabiya- Lebanon
P.O.Box: 70736 Antelias

**MEDICAL
DEVICE**

CE 0373

STERILE A